

INFORM Probes	List Number	NexES Software Version 9.30 Benchmark Procedure	賦活化			Other Options	Counterstain NFR	Counterstain or NFR Light	Fluorescence Counterstain	Positive Control Slide	
			Cell Conditioning	Enzyme	Incu. Time					List Number	
<b>INFORM HER-2/neu</b>											
HER-2/neu Probe	780-2840	INFORM HER-2/neu PLUS	Mild	ISH Protease 3	28	ISH Blocker	-	-	DAPI or PI	781-2840	Cell Lines
ISH Negative Control Probe	780-2902	INFORM HER-2/neu PLUS	Mild	ISH Protease 3	28	ISH Blocker	-	-	DAPI or PI	781-2840	Cell Lines
<b>INFORM CMV (use 760-092 ISH /VIEW Blue kit)</b>											
CMV Probe	780-2841	INFORM Probes iVIEW Blue	Standard	ISH Protease 1	4	-	4 min	4 min	-	782-2841	Cell Lines
RNA Positive Control Probe	780-2846	INFORM Probes iVIEW Blue	Standard	ISH Protease 1	4	-	4 min	4 min	-	782-2841	Cell Lines
INFORM Negative Control Probe	780-2847	INFORM Probes iVIEW Blue	Standard	ISH Protease 1	4	-	4 min	4 min	-	782-2841	Cell Lines
<b>INFORM EBER (use 760-092 ISH /VIEW Blue kit)</b>											
EBER Probe	780-2842	INFORM Probes iVIEW Blue	Standard	ISH Protease 1	4	-	4 min	4 min	-	783-2842	Cell Lines
RNA Positive Control Probe	780-2846	INFORM Probes iVIEW Blue	Standard	ISH Protease 1	4	-	4 min	4 min	-	783-2842	Cell Lines
INFORM Negative Control Probe	780-2847	INFORM Probes iVIEW Blue	Standard	ISH Protease 1	4	-	4 min	4 min	-	783-2842	Cell Lines
<b>INFORM Kappa (use 760-092 ISH /VIEW Blue kit)</b>											
Kappa Probe	780-2843	INFORM Probes iVIEW Blue	Mild	ISH Protease 3	24	-	4 min	4 min	-	782-2843	Cell Lines
RNA Positive Control Probe	780-2846	INFORM Probes iVIEW Blue	Mild	ISH Protease 3	24	-	4 min	4 min	-	782-2843	Cell Lines
INFORM Negative Control Probe	780-2847	INFORM Probes iVIEW Blue	Mild	ISH Protease 3	24	-	4 min	4 min	-	782-2843	Cell Lines
<b>INFORM Lambda (use 760-092 ISH /VIEW Blue kit)</b>											
Lambda Probe	780-2844	INFORM Probes iVIEW Blue	Mild	ISH Protease 3	24	-	4 min	4 min	-	782-2844	Cell Lines
RNA Positive Control Probe	780-2846	INFORM Probes iVIEW Blue	Mild	ISH Protease 3	24	-	4 min	4 min	-	782-2844	Cell Lines
INFORM Negative Control Probe	780-2847	INFORM Probes iVIEW Blue	Mild	ISH Protease 3	24	-	4 min	4 min	-	782-2844	Cell Lines
<b>INFORM HPV Tissue (use 760-097 ISH /VIEW Blue Plus kit)</b>											
HPV High Risk Probe(16)	780-4295	INFORM HPV III iVIEW Blue +	Extended	ISH Protease 3	4	-	NFR II = 4 min	-	-	783-2219	Cell Lines
HPV Low Risk Probe(6)	780-2220	INFORM HPV II iVIEW Blue +	Mild	ISH Protease 3	4	-	NFR II = 4 min	-	-	782-2839	Cell Lines
Alu Positive Control Probe II	780-2221	INFORM HPV II iVIEW Blue +	Mild	ISH Protease 3	4	-	NFR II = 4 min	-	-	783-2838 782-2839	Cell Lines
ISH Negative Control Probe	780-2902	INFORM HPV II iVIEW Blue +	Mild	ISH Protease 3	4	-	NFR II = 4 min	-	-	783-2838 782-2839	Cell Lines